



BT-Reveal™
Early Pancreatic
Cancer Test

Test Provided by
BREAKTHROUGH
GENOMICS

Instructions for Collection of Blood Sample

For Patient:

- No Fasting Required
- No Special Timing Required

For Nurse, Phlebotomist, or Healthcare Provider:

Specimen Type: **Whole Blood**

Collection Tube: **Use Tube Provided in Test Kit or Streck BCT or Nanograb Tube**

Collection Instructions:

1. **Fill tube completely (10ml)**
2. **Gently turn tube upside down (8-10 times)**
3. **Store and Ship at Room Temperature (do not refrigerate or heat)**

Please Note: Sample is stable for 7 days* at ambient temperature

Sample Label Instructions:

Please Print on Label:

Patient Name, Date of Birth, Collection Date

Shipping Instructions:

1. Place Tube with the absorbent pad in the biohazard bag and seal tightly
2. Place the biohazard bag into the padded envelope
3. Place the envelope into the return box along with the printed test requisition form (Printed forms are required only when tests have not yet been ordered through our secure online portal - order.btgenomics.com)
4. Close the box and place in the plastic overwrap
5. Write Sender Name and Address in the Upper Left Corner of the Overwrap or Use the Return Label Provided
6. Ship To:

Breakthrough Genomics
2 Hughes, Suite 100
Irvine, CA 92618

Ship at ambient temperature. Attach the return label on the package and ship.

If you have any questions,

please call [949-229-0094](tel:949-229-0094) or email us at test@btgenomics.com

* Samples received in the lab that were collected more than 5 days prior will not be processed